



SKIN FRICTION PROTECTORS

Small - 94201, Medium - 94204

Directions for Use:

Application:

- Clean and dry the area before applying (use alcohol prep pad or use other skin cleaning method).
- Peel the paper backing off completely.
- Firmly press Shear Points Skin Friction Protector over the skin area you want to protect.
- Use care when applying clothing over Shear Points Skin Friction Protector to prevent shifting.



Removal:

- Gently remove from skin by slowly peeling the Shear Point for a minimum of 5 seconds.
- Remove Shear Points Skin Friction Protector and reapply if wrinkles develop.

Precautions:

- ★ Do not apply Shear Points over current blisters, open skin, pre-existing skin conditions or thinning skin.
- ★ Gently and slowly remove from skin. Should Shear Points Skin Friction Protector become difficult to remove, use a mild lotion or skin sensitive oil to help with removal.
- If skin shows signs of new or increased irritation after application, remove, discontinue use and contact a physician or care provider.
- Inspect for damage. Should LiquiCell Shear Points become punctured and liquid contacts the eyes, flush immediately with clean water for 15 minutes. If irritation develops contact a physician or care provider.
- Do not reuse.



PH 952.937.7968 www.birchlabs.com

1307306

Made in U.S.A.
LiquiCell and Shear Points are registered
trademarks of Birchwood Laboratories LLC
Patent Pending.

